

Identifying the right patient for hereditary cancer testing

Do any of the following apply to your patient?

Patient history

- Cancer diagnosed under age 50
- Cancer diagnosed at any age in the context of significant family history (see right)
- Bilateral or multiple primary cancers
- Rare cancer^a
- Ashkenazi Jewish ancestry^b

Family history

- **≥2 relatives** with related cancer,^c with 1 diagnosed under age 50
- **≥3 relatives** with related cancer^c on the same side of the family
- Relative with a known familial mutation
- Ashkenazi Jewish ancestry^b

^a Male breast cancer, metastatic prostate cancer, ovarian cancer, pancreatic cancer, colorectal or uterine cancer with abnormal microsatellite instability/immunohistochemistry (MSI/IHC), pheochromocytoma, paraganglioma, 10 or more gastrointestinal polyps

^b Increased risk for specific *BRCA1* and *BRCA2* mutations

^c For a complete list of related cancer types, see the Hereditary Cancer Reference Guide

Choosing the right test: a quick guide to help with collaborative, informed decision-making

History consistent with multiple cancer syndromes **or** history not explained by previous genetic testing

Comprehensive Hereditary Cancer Panel

66 genes including high-risk, moderate-risk, and emerging genes associated with a broad spectrum of hereditary cancers

Guideline-Based Hereditary Cancer Panel

32 genes associated with a broad spectrum of hereditary cancers; all genes have management recommendations for 1 or more cancer sites

History includes primarily 1 cancer type

Targeted Panels

Hereditary Breast Cancer Panel:
18 genes associated with increased risk for breast cancer

Hereditary Colorectal Cancer Panel: 20 genes associated with increased risk for colorectal cancer

Hereditary Endocrine Cancer Panel: 12 genes associated with increased risk for paragangliomas, pheochromocytomas, and endocrine cancer

History is suspicious for a well-characterized hereditary cancer syndrome

Additional hereditary cancer risk tests

Syndrome-specific tests that analyze genes associated with well-characterized cancer syndromes, such as tuberous sclerosis complex (TSC), Lynch syndrome, familial adenomatous polyposis (FAP), and others

Patient has a relative with a familial mutation

Hereditary cancer, single-site test

Only looks for variant previously identified in a relative

For assistance, contact Genomic Client Services at **1.866.GENE.INFO (1.866.436.3463)** or **QuestHereditaryCancer.com**

The information presented is not intended to be a complete source for cancer risk in any patient. The patient's physician must determine the patient's risk based on the patient's clinical assessment, history, and the physician's training and experience.

QuestDiagnostics.com

Quest, Quest Diagnostics, any associated logos, and all associated Quest Diagnostics registered or unregistered trademarks are the property of Quest Diagnostics. All third-party marks—® and ™—are the property of their respective owners. © 2023 Quest Diagnostics Incorporated. All rights reserved. MI7552 3/2023