


Cardio IQ®


Managing Residual Risk with Advanced Cardiovascular Insights

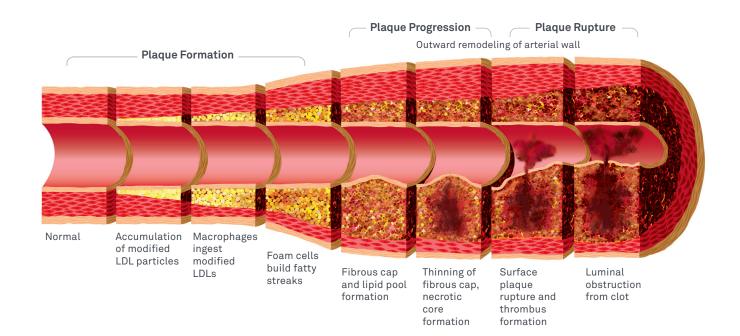

The Cardio IQ® Report — Key Attributes

Test results are shown in "Optimal," "Moderate," or "High" risk categories and are color-coded to display progressive risk values.

Historic results of previous tests are provided with the Cardio IQ report to help monitor patient progress.

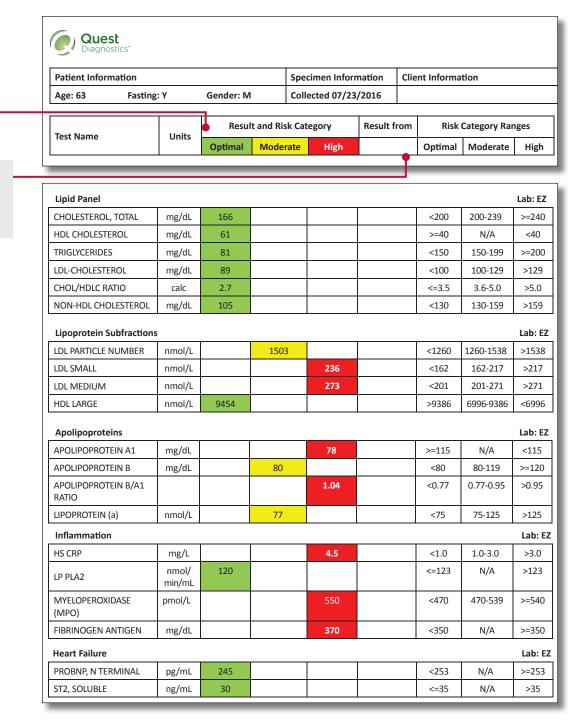


Note: This report example represents the entire portfolio of Quest Diagnostics Cardio IQ tests, and is not meant to represent a specific patient.

This report format is not available for patients under the age of 20 years.

Reference ranges are customized by gender and age for each patient when possible, based on data from published literature. Reference Range/Comments reported separately at the end of the report.

The Cardio IQ® Report — Key Attributes

(Continued)

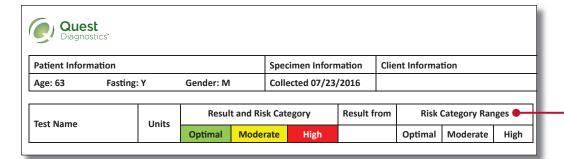

Risk ranges indicate **three levels of risk** based on data from published literature.

Metabolic Markers							Lab: EZ
HOMOCYSTEINE, CARDIOVASCULAR	umol/L			12.9	<11.4	N/A	>=11.4
HEMOGLOBIN A1c	% of total Hgb		5.8		<=5.6	5.7-6.4	>=6.5
VITAMIN D, 25-OH, TOTAL	ng/mL	86			>=30	20-29	<20
INSULIN	uIU/mL	10.0			<23	N/A	>=23
GLUCOSE	mg/dL		110		65-99	100-125	>=126

 Omega 3 & 6 Fatty Acids, Plasma
 Lab: EZ

 OMEGA 3 (EPA+DHA)
 %
 4.5

 INDEX
 >3.2
 2.2-3.2
 <2.2</td>

For details on reference ranges please refer to the reference range/comment section of the report.

Test Name	Result	Comments (See Guidance Statements)

Genetic Cardiovascular Markers Lab:

Genetic Cardiovascular Markers		Lau. CZ
LPA ASPIRIN GENOTYPE	lle/Met	Heterozygous carrier: associated with elevated Lp(a) levels and CVD risk and aspirin response in some clinical studies.
KIF6 GENOTYPE	Trp/Arg	Heterozygous carrier: associated with increased CHD risk and greater CHD event reduction with atorvastatin and pravastatin therapy in certain clinical studies.
9p21 GENOTYPE		Homozygous carrier (rs10757278 and rs133049). Increased 9p21 associated CVD risk.
rs10757278	gg	
rs1333049	сс	
APO E GENOTYPE	3/3	Apo E3 Carrier. Most common (normal) genotype.
LPA INTRON 25 GENOTYPE	tt	Homozygous noncarrier.
CYP2C19 GENOTYPE	*2	Intermediate Metabolizer.
4q25 AF RISK GENOTYPE		Noncarrier: No increased 4q25 associated risk of atrial fibrillation or cardioembolic stroke.
rs2200733	СС	
rs10033464	gg	

4myheart Diet & Exercise Coaching Program: Need help achieving and maintaining an optimal weight? Managing stress? Trying to improve physical fitness levels? The 4myheart program provides support and personalized lifestyle guidance to help improve heart health. Please talk to your provider, visit 4myheart.com or call 1.800.432.7889 option 2 to learn more.


Medical Information for Healthcare Providers: If you have any questions about any of the tests in our Cardio IQ offering, please call 1.800.432.7889 option 3 to speak to a clinical liaison. For frequently asked questions, you can also visit us at Education.QuestDiagnostics.com/FAQ/FAQ134

Advanced cardiovascular tests are organized into **functional categories** for ease of interpretation:

- Lipid Panel
- Lipoprotein Subfractionation
- Apolipoproteins
- Inflammation
- Heart Failure
- Metabolic Markers
- Genetic Cardiovascular Markers

The Cardio IQ® Report — Ion Mobility Detail

Lipid subclass distributions with related results are displayed graphically for ease of patient education

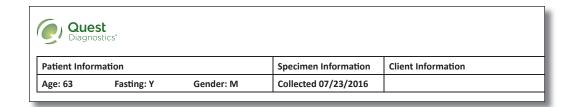

The **y-axis** of the graph shows total lipoprotein mass.

The **x-axis** of the graph shows HDL and LDL diameter in Angstroms (Å).


Shaded areas show key clinical management metrics. The height and width of the area under the curve indicate the amount of lipoprotein particles contained within each subclass.

HDL (High-Density Lipids)

The largest of the HDL particles is a surrogate marker for a functional cardioprotective reverse cholesterol transport mechanism. The goal of treatment for CVD risk reduction is to move from a predominance of small HDL particles to large HDL particles.

LDL (Low-Density Lipids)

The smaller subclasses of the LDL particles are associated with increased inflammatory potential, CVD risk, and rate of atherogenic progression. The goal of treatment for CVD risk reduction is to move from smaller to larger LDL particles.

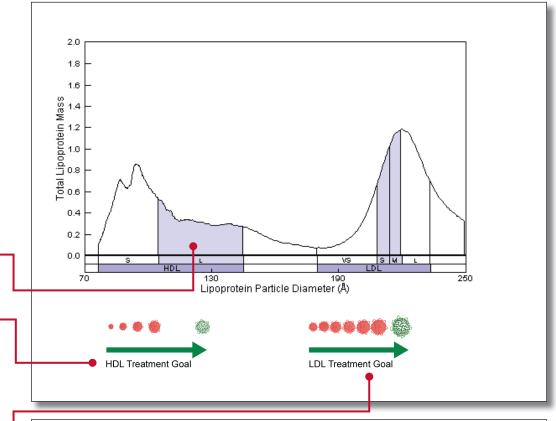



Test Name	Units	Result with Risk Category			Result from	Risk Category Ranges		
		Optimal	Moderate	High	Result from	Optimal	Moderate	High
Lipoprotein Subfractions								Lab: E
LDL PATTERN	Pattern	А				А	N/A	Lab: E

High tertile cut-points are based on a reference range population. Risk of CVD events is based on a reanalysis (unpublished) of the data presented in Musunuru et al. *ATVB* 2009;29:1975-80.

The Cardio IQ® Report — Other Detail


Metrics and information needed for comprehensive personalized reporting of cardiovascular risk

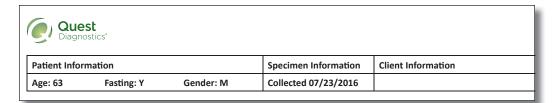

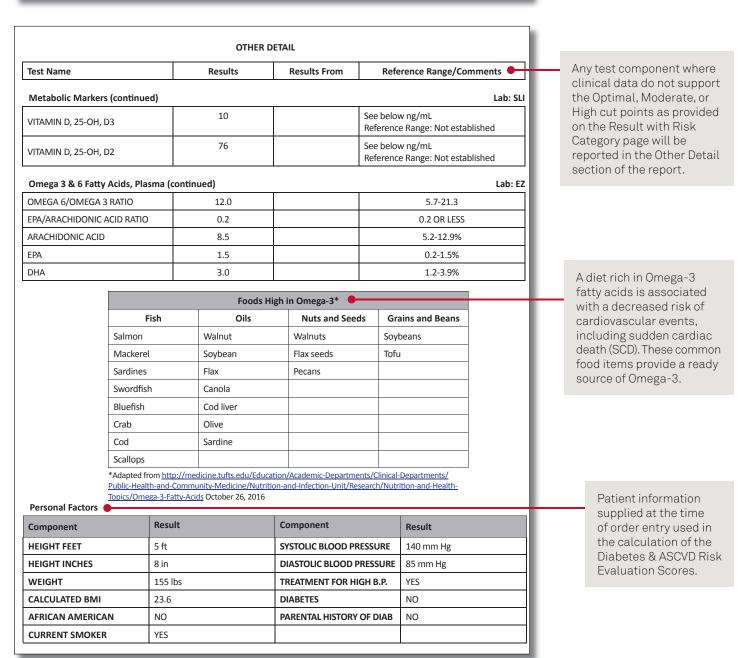


The Cardio IQ® Report — ASCVD Risk Evaluation

An expanded offering for insight into ASCVD risk is now part of the Cardio IQ Report

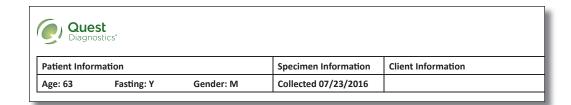


The Lipid Panel/ASCVD Risk Panel Assessment provides the 10-year and lifetime risk and 10-year goal of atherosclerotic cardiovascular disease (ASCVD) using lipid results with anthropomorphic data and family history.*

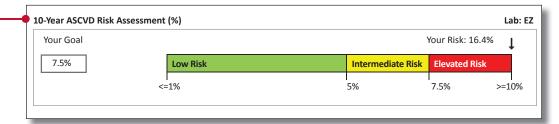

10-year ASCVD risk categories: > or =7.5% elevated risk; 5% to <7.5% intermediate risk; and <5% lower risk. Risk estimates for an ASCVD event (nonfatal MI, CHD death, or stroke) during the next 10 years are intended for patients currently free of clinical ASCVD; risk was estimated using the Pooled Cohort Equations: see guidelines for ethnic group-specific considerations. [Stone et al. Circulation 2013; Goff et al. Circulation 2013]

This patient-specific 10-year ASCVD risk goal is based on the patient's age, sex, ethnicity and optimal levels for other risk factors. This risk goal is calculated using the Pooled Cohort Equations. [Goff et al. Circulation 2013]

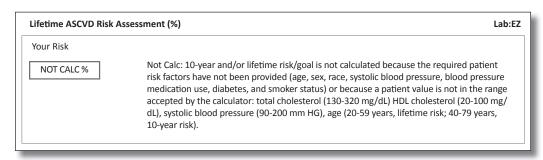


^{*}The ASCVD risk assessment is recommended in the 2013 ACC/AHA Guidelines: Treatment of Blood Cholesterol to Reduce Atherosclerotic Cardiovascular Risk in Adults.¹


Reference

[†]Risk calculations are provided at no additional cost and require the following personal factors at time of order entry: Age, Gender, Race, Systolic BP, Diastolic BP, Treatment for High Blood Pressure, Diabetes, Parental history of diabetes, Smoker.

Stone NJ, Robinson J, Lichtenstein AH, Bairey Merz CN, Blum CB, Eckel RH, Goldberg AC, Gordon D, Levy D, Lloyd-Jones DM, McBride P, Schwartz JS, Shero ST, Smith SC Jr, Watson K, Wilson PWF. 2013 ACC/AHA guidelines on the treatment of blood cholesterol to reduce atherosclerotic cardiovascular risk in adults: a report of the American College of Cardiology/American Heart Association Task Force on Practice Guidelines. 2013. This article is co-published in the Journal of the American College of Cardiology.

The Cardio IQ® Report — Diabetic Risk Evaluation

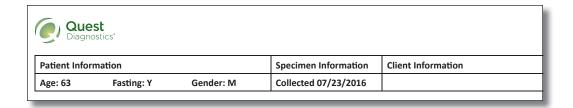

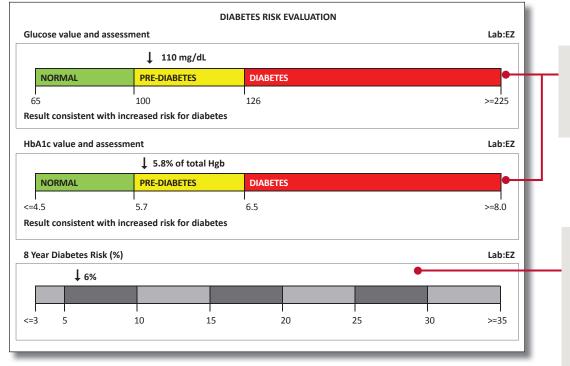

[‡]Risk calculations are provided at no additional cost and require the following personal factors at time of order entry: Age, Gender, Race, Systolic BP, Diastolic BP, Treatment for High Blood Pressure, Diabetes, Parental history of diabetes, Smoker.

The Metabolic Markers used in the Diabetes Risk Evaluation are visually presented to facilitate patient education.

The risk estimate is based upon the assessment of adults ages 30-79 in the Framingham study.² An estimate will not be provided if HbA1c and/or glucose indicate that the patient has diabetes. If fasting conditions were not met, use caution when interpreting the glucose test result and the risk estimate.[‡]

Reference

^{2.} Wilson PW, Meigs JB, Sullivan L, Fox CS, Nathan DM, D'Agostino RB Sr. Prediction of incident diabetes mellitus in middle-aged adults: The Framingham Offspring Study. Arch Intern Med. 2007; 167:1068-74.



The Cardio IQ® Solution

Offers a comprehensive approach to heart health

- Assess Baseline Risk by using advanced cardiovascular testing to help characterize a patient's individual cardiovascular disease risk.
- **Guide Personalized Therapy** by using a patient's unique makeup to help determine the therapy they need to achieve better heart health.
 - Initiate/intensify statin therapy
 - Identify opportunities for adjunct therapy
 - Set diet, exercise, and lifestyle targets
- Monitor Response to Therapy against a patient's testing history to hone their treatment plan.
- **Provide Ongoing Patient Support** through Clinical Educators that helps patients understand their test results and learn how they can adopt and adhere to their clinicians' treatment plans.
- Offer the 4myheart® Program, which is available at no additional cost for patients who have had testing performed. The Quest Diagnostics Clinical Educators work with patients to set goals, focus on lifestyle changes and develop treatment-adherence strategies to help reduce overall cardiovascular risk. For more information, visit 4myheart.com or call 1.800.432.7889.

For more information, contact your Quest Diagnostics sales representative or visit QuestDiagnostics.com/TestCenter